


# Lysil™

Silicone Gel Sheet for Scar Management

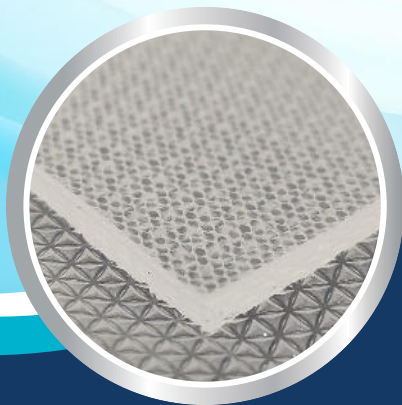

## PRODUCT DESCRIPTION:

**Lysil™** is composed of soft, semi occlusive polysiloxane material which has excellent bio-compatibility as per applicable EN ISO 10993 standards, extremely pure and reinforced with Polyamide-6 (Nylon) fabric which enhances the durability and handling properties and is permeable to water vapor. It is proved to promote hydration of stratum corneum of the scar area to eventually flatten and fade the scar. **Lysil™ Gel** sheet acts as a barrier for soft scar tissue to prevent bacterial entry on the sensitive scar tissues. **Lysil™ Silicone Gel** sheet greatly reduces itching and discomfort associated with scars.

## INTENDED USE/INDICATIONS:

Fresh or surgically re-visited

Raised hypertrophic -

Keloid scars -




## CONTRAINDICATIONS:

**Lysil™ Gel** sheet is not indicated for the treatment of depressed or pitted scars.

**Lysil™** should be applied as soon as the skin shows signs of complete epithelialization and or after surgical sutures have been removed if applicable.

Application of **Lysil™** to the contour surfaces or surface that are difficult to conceal is not recommended.

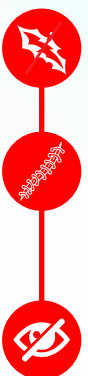

## DIRECTIONS FOR USE:



Wash hands before and after use.

1



Clean the scar and surrounding skin with a mild soap and rinse with warm water. Thoroughly dry the scar and surrounding skin.

2



Remove the **Lysil™** Gel sheet from the pack. with a non-fluffy/lint-free cloth.

3



For small scars, cut the **Lysil™** with clean scissors to fit scar size, with a small overlap over the surrounding skin.

4



Store any remaining part of **Lysil™** in a dry place (The packaging tray can be re-used).

5



Remove the plastic liner sheet carefully and without stretching.

6



Place adhesive side of the **Lysil™** to the scar, without stretching. If necessary a light bandage or tape can be used to keep the **Lysil™** in place.

7



Clean the **Lysil™** twice daily with a mild soap and rinse well and dry with non-fluffy / lint-free towel or by airing.

8

## PRECAUTIONS AND WARNINGS:

Care should be taken to ensure the cleanliness on injured area prior to application of **Lysil™** to prevent from any rashes due to poor hygiene practices.

Symptoms of rashes shall be overcome by discontinuing **Lysil™** for approximately 24 hours as rashes may be attributed by tight wrapping of gel sheet.

Discoloration may be experienced among dark skinned patients, when the area of skin is covered for extended period of time.

Not to be swallowed and keep out of reach of children.

## AVAILABILITY:


**Lysil™** is available in the following dimensions.




5 x 5cm



10 x 10cm

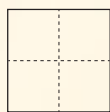

20 x 20cm

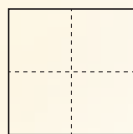

40 x 40cm

(on request)



Also available in Gel

## Eucare Pharmaceuticals (P) Ltd.,

Plot No. AC-25B, SIDCO Industrial Estate,  
Thirumudivakkam, Chennai - 600 132, INDIA  
Customer care No.: +91 94444 14801  
Email: info@eucareindia.com  
[www.eucareindia.com](http://www.eucareindia.com), [www.eucare.in](http://www.eucare.in)  
MADE IN INDIA